

Recommended protocol for SOX10

Obtained in run 60

19 Aug 2020

Immunostainer

Type: Dako Autostainer

Primary antibody

Clone: EP268
Producer: Cell Marque
Product no. / lot no.: 383R-14, -15 or -16 / 0000056862
Diluent: Antibody Diluent
Diluent producer / no.: Dako / K8006
Dilution factor: 1:400
Incubation time / temperature: 20 min. / 20°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Dako TRS High pH (3-1)
Heating time at max. temp.: 10 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako
Product / no: EnVision FLEX / K8000/SM802
Linker: Linker, Rabbit
Incubation time linker: 15 min.
Incubation time polymer: 20 min.
Incubation temperature: 20°C

Chromogen

Producer: Dako
Product / no: DAB+ / DM827 / DM823 / K800
Incubation time / temperature: 10 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.