

## Recommended protocol for PD-L1 (TPS/CPS)

Obtained in run C8

20 Aug 2020

### Immunostainer

Type: Lumatas Autostaining System Mo

### Primary antibody

Clone: MX070C  
Producer: maixin  
Product no. / lot no.: MAB-0854 / 2005090854A  
Diluent: None  
Diluent producer / no.: None / None  
Dilution factor: 1:0  
Incubation time / temperature: 30 min. / 32°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Std Tris-EDTA / EGTA pH 9  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 102°C

### Visualization system

Producer: maixin  
Product / no: DAB ( Titan ) / TT-0801  
Linker: None  
Incubation time polymer: 40 min.  
Incubation temperature: 25°C

### Chromogen

Producer: maixin  
Product / no: DAB ( Titan ) / TT-0801  
Incubation time / temperature: 6 min. / 25°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.