

Recommended protocol for MLA

Obtained in run 60

14 Sep 2020

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: EP43
Producer: Nordic Biosite
Product no. / lot no.: AC-0041 / 20040201
Diluent: Antibody Diluent
Diluent producer / no.: Dako / S2022
Dilution factor: 1:25
Incubation time / temperature: 20 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana Ultra CC1
Heating time at max. temp.: 36 min.
Maximum heating temp.: 100°C

Epitope retrieval, proteolysis

Enzyme: Protease 3
Enzyme producer / no: Ventana / 760-2020
Incubation time / temp: 4 min. / 36°C

Visualization system

Producer: Ventana
Product / no: UltraView Universal Alkaline Phosphatase Red Detection Kit
Incubation time polymer: 12 min.
Amplifier: Ultraview/iView Amplification Kit
Incubation time amplifier: 8+8 min.
Incubation temperature: 36°C

Chromogen

Producer: Ventana
Product / no: ultraView Universal Alkaline Phosphatase Red Detection Kit
Incubation time / temperature: 20 min. / 36°C
Enhancement: None



Nordic Immunohistochemical Quality Control

Institute of Pathology, Aalborg University Hospital, Ladegaardsgade 3, P.O.Box 561, DK-9100 Aalborg, Denmark

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.