

Recommended protocol for PD-L1 (TPS/CPS)

Obtained in run C8

31 Aug 2020

Immunostainer

Type: Dako Autostainer Link 48 +

Primary antibody

Clone: 22C3
Producer: Dako
Product no. / lot no.: SK006 / 11101640B
Format: Ready-To-Use (prediluted)
Incubation time / temperature: 30 min. / 22°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Dako TRS Low pH (3-1)
Heating time at max. temp.: 20 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako
Product / no: PD-L1 IHC 22c3 pharmDx / SK006
Incubation time linker: 30 min.
Incubation time polymer: 30 min.
Incubation temperature: 22°C

Chromogen

Producer: Dako
Product / no: PD-L1 IHC 22c3 pharmDx / SK006
Incubation time / temperature: 10 min. / 22°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.