

## Recommended protocol for CD15

Obtained in run 61

03 Dec 2020

### Immunostainer

Type: Dako Omnis

### Primary antibody

Clone: Carb-3  
Producer: Dako/Agilent  
Product no. / lot no.: GA062 / 11178404  
Format: Ready-To-Use (prediluted)  
Incubation time / temperature: 20 min. / 31°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Dako Omnis Target Retrieval Solution, High pH  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 97°C

### Visualization system

Producer: Dako Omnis  
Product / no: EnVision Flex / GV800/GV823  
Linker: None  
Incubation time polymer: 22 min.  
Incubation temperature: 31°C

### Chromogen

Producer: Dako Omnis  
Product / no: DAB+ Substrate Chromogen System / GV825  
Incubation time / temperature: 5 min. / 31°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.