

Recommended protocol for HER2 IHC

Obtained in run B31

10 Dec 2020

Immunostainer

Type: Dako Omnis

Primary antibody

Clone: Polyclonal
Producer: Dako
Product no. / lot no.: A0485 / 20073789
Diluent: Antibody Diluent
Diluent producer / no.: Dako / K8006
Dilution factor: 1:200
Incubation time / temperature: 20 min. / 32°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Dako Omnis Target Retrieval Solution, Low pH
Heating time at max. temp.: 30 min.
Maximum heating temp.: 32°C

Visualization system

Producer: Dako Omnis
Product / no: EnVision Flex / GV800/GV823
Linker: Rabbit LINKER
Incubation time linker: 10 min.
Incubation time polymer: 20 min.
Incubation temperature: 32°C

Chromogen

Producer: Dako Omnis
Product / no: DAB+ Substrate Chromogen System / GV825
Incubation time / temperature: 5 min. / 32°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.