

## Recommended protocol for p63

Obtained in run 61

15 Dec 2020

### Immunostainer

Type: Ventana Benchmark Ultra

### Primary antibody

Clone: 4A4  
Producer: Biocare Medical  
Product no. / lot no.: CM 163 A, B, C / 031720A  
Diluent: Antibody Diluent  
Diluent producer / no.: Ventana / 251-018  
Dilution factor: 1:100  
Incubation time / temperature: 32 min. / 36°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Ventana Ultra CC1  
Heating time at max. temp.: 64 min.  
Maximum heating temp.: 95°C

### Visualization system

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Incubation temperature: 36°C

### Chromogen

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time / temperature: 8 min. / 36°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.