

## Recommended protocol for p63

Obtained in run 61

17 Dec 2020

### Immunostainer

Type: Gene Tech Genestainer

### Primary antibody

Clone: GR004  
Producer: Gene Tech  
Product no. / lot no.: GT2331 / 2020112601  
Diluent: Ab diluent  
Dilution factor: 1:100  
Incubation time / temperature: 50 min. / 25°C

### Epitope retrieval, HIER

Device: Water bath  
Buffer:  
Heating time at max. temp.: 15 min.  
Maximum heating temp.: 95°C

### Visualization system

Producer: Gene Tech  
Product / no: GTVision / GK8007  
Incubation time polymer: 20 min.  
Incubation temperature: 25°C

### Chromogen

Producer: Gene Tech  
Product / no: DAB / GK8007  
Incubation time / temperature: 5 min. / 25°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.