

## Recommended protocol for CD15

Obtained in run 61

22 Dec 2020

### Immunostainer

Type: Dako Autostainer

### Primary antibody

Clone: Carb-3  
Producer: Dako  
Product no. / lot no.: M3631 / 10109709  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / S0809  
Dilution factor: 1:50  
Incubation time / temperature: 60 min. / 22°C

### Epitope retrieval, HIER

Device: PT-link / PT-module  
Buffer: Dako TRS High pH (3-1)  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 97°C

### Visualization system

Producer: Dako  
Product / no: EnVision FLEX / K8000/SM802  
Linker: None  
Incubation time polymer: 30 min.  
Incubation temperature: 22°C

### Chromogen

Producer: Dako  
Product / no: DAB+ / DM827 / DM823 / K800  
Incubation time / temperature: 10 min. / 22°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.