

Recommended protocol for HER2 IHC

Obtained in run B31

29 Dec 2020

Immunostainer

Type: Ventana Benchmark XT

Primary antibody

Clone: Polyclonal
Producer: Dako
Product no. / lot no.: A0485 / 20062529
Diluent: Antibody Diluent
Diluent producer / no.: Ventana / 251-018
Dilution factor: 1:300
Incubation time / temperature: 28 min. / 25°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana Ultra CC1
Heating time at max. temp.: 30 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Ventana
Product / no: UltraView Universal DAB Detection Kit / 760-500
Incubation time polymer: 8 min.
Amplifier: Ultraview/iView Amplification Kit
Incubation time amplifier: 4+8 min.
Incubation temperature: 37°C

Chromogen

Producer: Ventana
Product / no: ultraView Universal DAB Detection Kit / 760-500
Incubation time / temperature: 8 min. / 37°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.