

Nordic Immunohistochemical Quality Control

Institute of Pathology, Aalborg University Hospital, Ladegaardsgade 3, P.O.Box 561, DK-9100 Aalborg, Denmark

Recommended protocol for MSH6

Obtained in run 61

30 Dec 2020

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: SP93

Producer: Cell Marque

Product no. / lot no.: 287R-24/25/26 / 86841

Diluent: Antibody Diluent

Diluent producer / no.: Dako / K8006

Dilution factor: 1:100

Incubation time / temperature: 32 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine

Buffer: Ventana CC1

Heating time at max. temp.: 64 min. Maximum heating temp.: 98°C

Visualization system

Producer: Ventana

Product / no: OptiView DAB IHC Detection Kit / 760-700

Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 36°C

Chromogen

Producer: Ventana

Product / no: OptiView DAB IHC Detection Kit / 760-700

Incubation time / temperature: 4 min. / 36°C

Enhancement: CuSO4

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.