

## Recommended protocol for CD31

Obtained in run 62

09 Feb 2021

### Immunostainer

Type: Ventana Benchmark Ultra

### Primary antibody

Clone: JC70A  
Producer: Ventana  
Product no. / lot no.: 760-4378 / V0002234  
Format: Ready-To-Use (prediluted)  
Incubation time / temperature: 28 min. / 37°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Ventana Ultra CC1  
Heating time at max. temp.: 64 min.  
Maximum heating temp.: 95°C

### Visualization system

Producer: Ventana  
Product / no: UltraView Universal DAB Detection Kit / 760-500  
Incubation time polymer: 8 min.  
Amplifier: Ultraview/iView Amplification Kit  
Incubation time amplifier: 8+8 min.  
Incubation temperature: 36°C

### Chromogen

Producer: Ventana  
Product / no: ultraView Universal DAB Detection Kit / 760-500  
Incubation time / temperature: 8 min. / 36°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.