

## Recommended protocol for HER2 IHC

Obtained in run B31

10 Feb 2021

### Immunostainer

Type: Dako Autostainer Link 48 +

### Primary antibody

Clone: Polyclonal  
Producer: Dako  
Product no. / lot no.: A0485 / 20073789  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / K8006  
Dilution factor: 1:200  
Incubation time / temperature: 20 min. / 21°C

### Epitope retrieval, HIER

Device: PT-link / PT-module  
Buffer: Dako TRS Low pH (3-1)  
Heating time at max. temp.: 10 min.  
Maximum heating temp.: 97°C

### Visualization system

Producer: Dako  
Product / no: EnVision FLEX / K8000/SM802  
Linker: Linker, Rabbit  
Incubation time linker: 15 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 21°C

### Chromogen

Producer: Dako  
Product / no: DAB+ / DM827 / DM823 / K800  
Incubation time / temperature: 10 min. / 21°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.