

Recommended protocol for PAX8

Obtained in run 62

17 Feb 2021

Immunostainer

Type: Intellipath FLX

Primary antibody

Clone: BC12
Producer: Biocare Medical
Product no. / lot no.: API438 / 120120A
Diluent: None
Dilution factor: 1:1
Incubation time / temperature: 30 min. / 25°C

Epitope retrieval, HIER

Device: Pressure Cooker
Buffer: Biocare Medical Diva Decloker
Heating time at max. temp.: 15 min.
Maximum heating temp.: 110°C

Visualization system

Producer: Biocare Medical
Product / no: MACH 4 HRP Polymer / MHR534
Linker: MACH 4 Mouse Probe
Incubation time linker: 10 min.
Incubation time polymer: 20 min.
Incubation temperature: 25°C

Chromogen

Producer: Biocare Medical
Product / no: intelliPATH DAB / IPK5010
Incubation time / temperature: 5 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.