

## Recommended protocol for PAX8

Obtained in run 62

18 Feb 2021

### Immunostainer

Type: Dako Omnis

### Primary antibody

Clone: EP298  
Producer: Epitomics  
Product no. / lot no.: AC-0296 / EP060701  
Diluent: Renoir Red Diluent  
Dilution factor: 1:150  
Incubation time / temperature: 30 min. / 21°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Dako Omnis Target Retrieval Solution, High pH  
Heating time at max. temp.: 40 min.  
Maximum heating temp.: 97°C

### Visualization system

Producer: Dako Omnis  
Product / no: EnVision Flex / GV800/GV823  
Linker: Rabbit LINKER  
Incubation time linker: 10 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 21°C

### Chromogen

Producer: Dako Omnis  
Product / no: DAB+ Substrate Chromogen System / GV825  
Incubation time / temperature: 5 min. / 21°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.