

Recommended protocol for CK20

Obtained in run 62

23 Feb 2021

Immunostainer

Type: Gene Tech Genestainer

Primary antibody

Clone: KS20.8
Producer: Cell Marque
Product no. / lot no.: CMA-320M-16 / 51892
Diluent: Ab diluent
Diluent producer / no.: Gene Tech / GT100910
Dilution factor: 1:300
Incubation time / temperature: 50 min. / 25°C

Epitope retrieval, HIER

Device: Water bath
Buffer:
Heating time at max. temp.: 20 min.
Maximum heating temp.: 95°C

Visualization system

Producer: Gene Tech
Product / no: GTVision / GK8007
Incubation time polymer: 20 min.
Incubation temperature: 25°C

Chromogen

Producer: Gene Tech
Product / no: DAB / GK8007
Incubation time / temperature: 5 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.