

Recommended protocol for HER2 IHC

Obtained in run

15 Apr 2021

Immunostainer

Type: Dako Omnis

Primary antibody

Clone: DG44
Producer: Dako/Agilent
Product no. / lot no.: GE001 / NA
Format: Ready-To-Use (prediluted)
Incubation time / temperature: 10 min. / 32°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Dako Omnis Target Retrieval Solution, Low pH
Heating time at max. temp.: 30 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako/Agilent
Product / no: HercepTest mAb pharmDx Visualization Reagent / GE001
Linker: None
Incubation time polymer: 20 min.
Incubation temperature: 32°C

Chromogen

Producer: Dako/Agilent
Product / no: HercepTest mAb pharmDx EnVision FLEX DAB+ Chromogen
Incubation time / temperature: 5 min. / 32°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.