

## Recommended protocol for CD79a

Obtained in run 63

23 Jul 2021

### Immunostainer

Type: Thermo Autostainer 36/48/72

### Primary antibody

Clone: BSR20  
Producer: Nordic BioSite  
Product no. / lot no.: BSH-3007 / BSH1-3bd  
Diluent: Antibody Diluent  
Diluent producer / no.: ZYTOMED systems / ZUC025-500  
Dilution factor: 1:100  
Incubation time / temperature: 30 min. / 21°C

### Epitope retrieval, HIER

Device: PT-link / PT-module  
Buffer: Std Tris-EDTA / EGTA pH 9  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 98°C

### Visualization system

Producer: ZYTOMED systems  
Product / no: (HRP) Polymer anti-Rabbit / ZUC032-100  
Linker: None  
Incubation time polymer: 30 min.  
Incubation temperature: 21°C

### Chromogen

Producer: ZYTOMED systems  
Product / no: DAB substrate High contrast kit / DAB5000PLUS  
Incubation time / temperature: 10 min. / 21°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.