

Nordic Immunohistochemical Quality Control

Institute of Pathology, Aalborg University Hospital, Ladegaardsgade 3, P.O.Box 561, DK-9100 Aalborg, Denmark

Recommended protocol for GATA3

Obtained in run 63

19 Aug 2021

Immunostainer

Type: Gene Tech Genestainer

Primary antibody

Clone: EP368

Producer: Cell Marque

Product no. / lot no.: AC-0324A / 20092103

Diluent: Ab diluent

Diluent producer / no.: Gene Tech / GT100910

Dilution factor: 1:100

Incubation time / temperature: 50 min. / 25°C

Epitope retrieval, HIER

Device: Water bath

Buffer: Std Tris-EDTA / EGTA pH 9

Heating time at max. temp.: 20 min. Maximum heating temp.: 95°C

Visualization system

Producer: Gene Tech

Product / no: GTVision / GK8007

Linker: None Incubation time polymer: 20 min. Incubation temperature: 25°C

Chromogen

Producer: Gene Tech
Product / no: DAB / GK8007
Incubation time / temperature: 5 min. / 25°C

Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.