

Recommended protocol for CK5

Obtained in run 63

23 Aug 2021

Immunostainer

Type: Dako Autostainer

Primary antibody

Clone: XM26
Producer: Leica/novocastra
Product no. / lot no.: NCL-L-CK5 / 6081348
Diluent: Antibody Diluent
Dilution factor: 1:100
Incubation time / temperature: 20 min. / 21°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Dako TRS High pH (3-1)
Heating time at max. temp.: 10 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako
Product / no: EnVision FLEX / K8000/SM802
Linker: Linker, Mouse
Incubation time linker: 15 min.
Incubation time polymer: 20 min.
Incubation temperature: 21°C

Chromogen

Producer: Dako
Product / no: DAB+ / DM827 / DM823 / K800
Incubation time / temperature: 10 min. / 21°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.