

Recommended protocol for Bcl-2

Obtained in run

26 Sep 2015

Immunostainer

Type: Dako Autostainer

Primary antibody

Clone: 100
Producer: BioGenex
Product no. / lot no.: MU287A-UC / MU2870406
Diluent: Antibody Diluent
Dilution factor: 1:200
Incubation time / temperature: 30 min. / 100°C

Epitope retrieval, HIER

Device: Other
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 30 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Dako
Product / no: EnVision FLEX+ / K8012/DM822
Linker: None
Incubation time polymer: 30 min.
Incubation temperature: 20°C

Chromogen

Producer: Dako
Product / no: DAB+ / DM827 / DM823 / K800
Incubation time / temperature: 10 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.