

## Recommended protocol for GATA3

Obtained in run 63

30 Aug 2021

### Immunostainer

Type: Dako Autostainer

### Primary antibody

Clone: EP368  
Producer: Cell Marque  
Product no. / lot no.: AC-0324A / 19042608  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / K8006  
Dilution factor: 1:100  
Incubation time / temperature: 30 min. / 25°C

### Epitope retrieval, HIER

Device: PT-link / PT-module  
Buffer: Dako TRS High pH (3-1)  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 97°C

### Visualization system

Producer: Dako  
Product / no: EnVision FLEX+ / K8002/SM802  
Linker: Linker, Mouse  
Incubation time linker: 10 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 25°C

### Chromogen

Producer: Dako  
Product / no: DAB+ / DM827 / DM823 / K800  
Incubation time / temperature: 10 min. / 25°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.