

## **Nordic Immunohistochemical Quality Control**

Institute of Pathology, Aalborg University Hospital, Ladegaardsgade 3, P.O.Box 561, DK-9100 Aalborg, Denmark

## Recommended protocol for CD79a

## Obtained in run 63

31 Aug 2021

**Immunostainer** 

Type: Gene Tech Genestainer

**Primary antibody** 

Clone: GM101

Producer: Gene Tech

Product no. / lot no.: GT232502 / 2021042301

Diluent: None

Diluent producer / no.: None / None

Dilution factor: 1:0

Incubation time / temperature: 50 min. / 25°C

Epitope retrieval, HIER

Device: Water bath

Buffer: Std Tris-EDTA / EGTA pH 9

Heating time at max. temp.: 20 min. Maximum heating temp.: 95°C

Visualization system

Producer: Gene Tech

Product / no: GTVision / GK8007

Linker: None Incubation time polymer: 20 min. Incubation temperature: 25°C

Chromogen

Producer: Gene Tech
Product / no: DAB / GK8007
Incubation time / temperature: 5 min. / 25°C

Enhancement: None

## Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.