

Recommended protocol for EMA

Obtained in run

26 Sep 2015

Immunostainer

Type: Dako Autostainer

Primary antibody

Clone: E29
Producer: Dako
Product no. / lot no.: M0613 / 00041785
Diluent: Antibody Diluent
Dilution factor: 1:500
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Dako TRS High pH (3-1)
Heating time at max. temp.: 20 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako
Product / no: REAL EnVision Rabbit/Mouse / K5007 / K4065
Incubation time polymer: 30 min.
Incubation temperature: 20°C

Chromogen

Producer: Dako
Product / no: DAB+ / K5007 / K4065
Incubation time / temperature: 7 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.