

## Recommended protocol for CD3

Obtained in run 63

09 Sep 2021

### Immunostainer

Type: Ventana Benchmark Ultra

### Primary antibody

Clone: LN10  
Producer: Leica Biosystems  
Product no. / lot no.: NCL-L-CD3-565 / 6078812  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / K8006  
Dilution factor: 1:100  
Incubation time / temperature: 32 min. / 36°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Ventana Ultra CC1  
Heating time at max. temp.: 48 min.  
Maximum heating temp.: 99°C

### Visualization system

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Incubation temperature: 36°C

### Chromogen

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time / temperature: 8 min. / 36°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.