

## Recommended protocol for TTF1

Obtained in run

26 Sep 2015

### Immunostainer

Type: IntelliPath, Biocare

### Primary antibody

Clone: SPT24  
Producer: Leica/novocastra  
Product no. / lot no.: NCL-L-TTF-1 / 6022606  
Diluent: Da Vinci Green  
Dilution factor: 1:100  
Incubation time / temperature: 30 min. / 20°C

### Epitope retrieval, HIER

Device: Pressure Cooker  
Buffer: Biocare Diva pH 6  
Heating time at max. temp.: 15 min.  
Maximum heating temp.: 110°C

### Visualization system

Producer: Biocare  
Product / no: NS / Mach4  
Incubation time polymer: 30 min.  
Incubation temperature: 20°C

### Chromogen

Producer: Biocare  
Product / no: DAB / Mach4  
Incubation time / temperature: 5 min. / 20°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.