

## Recommended protocol for CD56

Obtained in run

27 Sep 2015

### Immunostainer

Type: Autostainer 720, LabVision

### Primary antibody

Clone: 56C04  
Producer: Master Diagnostica  
Product no. / lot no.: MAD-000218QD / 0149-01  
Format: Ready-To-Use (prediluted)  
Incubation time / temperature: 10 min. / 20°C

### Epitope retrieval, HIER

Device: PT-link / PT-module  
Buffer: Std EDTA / EGTA pH 8  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 95°C

### Visualization system

Producer: NS  
Product / no: NS / MAD-021881QK  
Linker: NS  
Incubation time linker: 10 min.  
Incubation time polymer: 10 min.  
Incubation temperature: 20°C

### Chromogen

Producer: NS  
Product / no: DAB / MAD-001811QK  
Incubation time / temperature: 5 min. / 20°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.