

Recommended protocol for HEPA

Obtained in run

27 Sep 2015

Immunostainer

Type: Teecan Freedom Evo

Primary antibody

Clone: OCH1E5
Producer: Dako
Product no. / lot no.: M7158 / 00072939
Diluent: Antibody Diluent
Diluent producer / no.: Dako / K8006
Dilution factor: 1:40
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Std Citrate pH 6
Heating time at max. temp.: 30 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Immunologic
Product / no: NS / DPVO-999HRP
Incubation time polymer: 30 min.
Incubation temperature: 20°C

Chromogen

Producer: Dako
Product / no: DAB+ / K3467 / K3468
Incubation time / temperature: 5 min. / 20°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.