

Recommended protocol for CD56

Obtained in run 64

16 Dec 2021

Immunostainer

Type: Dako Autostainer

Primary antibody

Clone: 123C3
Producer: Dako/Agilent
Product no. / lot no.: IR628/IS628 / IR628/Lot 20081080
Format: Ready-To-Use (prediluted)
Incubation time / temperature: 30 min. / 23°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Dako TRS High pH (3-1)
Heating time at max. temp.: 20 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako
Product / no: EnVision FLEX+ / K8002/SM802
Linker: Linker, Mouse
Incubation time linker: 15 min.
Incubation time polymer: 20 min.
Incubation temperature: 23°C

Chromogen

Producer: Dako
Product / no: DAB+ / DM827 / DM823 / K800
Incubation time / temperature: 10 min. / 23°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.