

Recommended protocol for CD20

Obtained in run

27 Sep 2015

Immunostainer

Type: Manual

Primary antibody

Clone: EP7
Producer: Epitomics
Product no. / lot no.: AC-0012 / TDB
Diluent: NS
Dilution factor: 1:100
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: Pressure Cooker
Buffer: Std Citrate pH 6
Heating time at max. temp.: 1 min.
Maximum heating temp.: 125°C

Visualization system

Producer: Epitomics
Product / no: NS / DK-0001
Incubation time polymer: min.
Incubation temperature: 20°C

Chromogen

Producer: Epitomics
Product / no: DAB / SP-0008
Incubation time / temperature: 10 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.