

Recommended protocol for CK20

Obtained in run

27 Sep 2015

Immunostainer

Type: Autostainer, LabVision

Primary antibody

Clone: Ks20.8
Producer: Dako
Product no. / lot no.: M7019 / 00065714
Diluent: 1% BSA in TBS
Dilution factor: 1:200
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: Microwave oven
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 15 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Thermo
Product / no: NS / TL-125-QHD
Linker: None
Incubation time polymer: 10 min.
Incubation temperature: 20°C

Chromogen

Producer: Thermo
Product / no: DAB / TL-125-QHD
Incubation time / temperature: 10 min. / 20°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.