

Recommended protocol for CD30

Obtained in run 65

16 Feb 2022

Immunostainer

Type: Thermo Autostainer 36/48/72

Primary antibody

Clone: Ber-H2
Producer: Epredia
Product no. / lot no.: MS-361-S / 361S2003F
Diluent: Normal Antibody Diluent
Diluent producer / no.: Immunologic / VWRKBD09-999
Dilution factor: 1:40
Incubation time / temperature: 60 min. / 25°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 99°C

Visualization system

Producer: Immunologic
Product / no: Mouse/ rabbit HRP / VWRKUBVB999HRP-POLYM
Linker: None
Incubation time polymer: 30 min.
Incubation temperature: 25°C

Chromogen

Producer: Immunologic
Product / no: Bright Dab Solution B / VWRKUBVB999HRP
Incubation time / temperature: 8 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.