

Recommended protocol for SOX10

Obtained in run

15 Jan 2016

Immunostainer

Type: IntelliPath, Biocare

Primary antibody

Clone: BC34
Producer: Biocare
Product no. / lot no.: AC13099C / 010815
Diluent: Antibody diluent
Dilution factor: 1:100
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: Pressure Cooker
Buffer: Biocare Diva Decloaker pH 6,2
Heating time at max. temp.: 5 min.
Maximum heating temp.: 110°C

Visualization system

Producer: Biocare
Product / no: NS / M4U534L
Linker: NS
Incubation time linker: 10 min.
Incubation time polymer: 15 min.
Incubation temperature: 20°C

Chromogen

Producer: Biocare
Product / no: DAB / M4U534L
Incubation time / temperature: 5 min. / 20°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.