

## Recommended protocol for CD79a

Obtained in run

17 Jan 2016

### Immunostainer

Type: Ventana Benchmark Ultra

### Primary antibody

Clone: SP18  
Producer: Cell Marque  
Product no. / lot no.: 179R-16 / 1200505B  
Diluent: Antibody Diluent with Casein  
Dilution factor: 1:300  
Incubation time / temperature: 24 min. / 20°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Ventana Ultra CC1  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 90°C

### Visualization system

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Incubation temperature: 36°C

### Chromogen

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time / temperature: 10 min. / 20°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.