

## Recommended protocol for SYP

Obtained in run 66

23 Aug 2022

### Immunostainer

Type: Dako Omnis

### Primary antibody

Clone: BS15  
Producer: Nordic BioSite  
Product no. / lot no.: BSH-7385 / BSH-8ak  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / K8006  
Dilution factor: 1:200  
Incubation time / temperature: 30 min. / 36°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Dako Omnis Target Retrieval Solution, High pH  
Heating time at max. temp.: 24 min.  
Maximum heating temp.: 97°C

### Visualization system

Producer: Dako Omnis  
Product / no: EnVision Flex / GV800/GV823  
Linker: Mouse LINKER  
Incubation time linker: 10 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 36°C

### Chromogen

Producer: Dako Omnis  
Product / no: DAB+ Substrate Chromogen System / GV825  
Incubation time / temperature: 5 min. / 36°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.