

## Recommended protocol for SYP

Obtained in run 66

29 Aug 2022

### Immunostainer

Type: LYNX480

### Primary antibody

Clone: BP6053  
Producer: Biolynx  
Product no. / lot no.: I10412E / B22032105  
Diluent: None  
Dilution factor: 1:0  
Incubation time / temperature: 30 min. / 25°C

### Epitope retrieval, HIER

Device: Other  
Buffer: Biolynx Antigen Retrieval 2 (EDTA)  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 100°C

### Visualization system

Producer: Biolynx  
Product / no: BXV Visualization System / I20032C  
Linker: Post Primary Antibody  
Incubation time linker: 10 min.  
Incubation time polymer: 10 min.  
Incubation temperature: 25°C

### Chromogen

Producer: Biolynx  
Product / no: DAB / I20032C  
Incubation time / temperature: 10 min. / 25°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.