

## **Recommended protocol for CK-LMW**

Obtained in run 49

28 Dec 2016

### **Immunostainer**

Type: Manual

### **Primary antibody**

Clone: EP17  
Producer: Epitomics  
Product no. / lot no.: AC-0007 / EM120405  
Diluent: Antibody Diluent  
Dilution factor: 1:100  
Incubation time / temperature: 30 min. / 25°C

### **Epitope retrieval, HIER**

Device: PT-link / PT-module  
Buffer: Diagnostic Bio Syste Tris-EDTA RB pH9,0  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 96°C

### **Visualization system**

Producer: Dako  
Product / no: REAL EnVision Rabbit/Mouse / K5007 / K4065  
Incubation time polymer: 30 min.  
Incubation temperature: 25°C

### **Chromogen**

Producer: Dako  
Product / no: DAB+ / K5007 / K4065  
Incubation time / temperature: 5 min. / 25°C  
Enhancement: None

### **Disclaimer:**

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.