

## Recommended protocol for MLH1

Obtained in run 67

07 Dec 2022

### Immunostainer

Type: Ventana Benchmark Ultra

### Primary antibody

Clone: G168-15  
Producer: BD Pharmigen  
Product no. / lot no.: 550838 / 0265997  
Diluent: Antibody Diluent with Casein  
Diluent producer / no.: Ventana / 760-219  
Dilution factor: 1:10  
Incubation time / temperature: 64 min. / 36°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Ventana Ultra CC1  
Heating time at max. temp.: 64 min.  
Maximum heating temp.: 95°C

### Visualization system

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Amplifier: OptiView Amplification Kit  
Incubation time amplifier: 4+4 min.  
Incubation temperature: 36°C

### Chromogen

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time / temperature: 8 min. / 36°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect



## **Nordic Immunohistochemical Quality Control**

Institute of Pathology, Aalborg University Hospital, Ladegaardsgade 3, P.O.Box 561, DK-9100 Aalborg, Denmark

the most recent developments in immunohistochemistry at any point in time.