

## Recommended protocol for URO II/III

Obtained in run 68

16 Feb 2023

### Immunostainer

Type: Ventana Benchmark Ultra

### Primary antibody

Clone: Uroplakin II BC21  
Producer: Zytomed  
Product no. / lot no.: MSG102 / A3991  
Diluent: None  
Diluent producer / no.: None / None  
Dilution factor: 1:0  
Incubation time / temperature: 60 min. / 37°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Ventana CC1  
Heating time at max. temp.: 64 min.  
Maximum heating temp.: 95°C

### Visualization system

Producer: Ventana  
Product / no: UltraView Universal DAB Detection Kit / 760-500  
Incubation time polymer: 8 min.  
Incubation temperature: 36°C

### Chromogen

Producer: Ventana  
Product / no: ultraView Universal DAB Detection Kit / 760-500  
Incubation time / temperature: 8 min. / 36°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.