

## Recommended protocol for URO II/III

Obtained in run 68

24 Mar 2023

### Immunostainer

Type: intelliPATH (Biocare)

### Primary antibody

Clone: BC21 + BC17  
Producer: Biocare Medical  
Product no. / lot no.: API 3094 AA / 102821A  
Format: Ready-To-Use (prediluted)  
Incubation time / temperature: 30 min. / 20°C

### Epitope retrieval, HIER

Device: Other  
Buffer: Zytomed Systems HIER Citrate pH6  
Heating time at max. temp.: 7 min.  
Maximum heating temp.: 110°C

### Visualization system

Producer: Zytomed Systems  
Product / no: ZytoChem Plus (HRP) Polymer Kit / POLHRP100  
Linker: Postblock  
Incubation time linker: 20 min.  
Incubation time polymer: 30 min.  
Incubation temperature: 20°C

### Chromogen

Producer: Zytomed Systems  
Product / no: DAB High Contrast / DABPLUS-5000  
Incubation time / temperature: 10 min. / 20°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.