

## Recommended protocol for URO II/III

Obtained in run 68

15 Mar 2023

### Immunostainer

Type: Titan S

### Primary antibody

Clone: MX130  
Producer: Fuzhou Maixin  
Product no. / lot no.: MAB-1099 / 2302221099a  
Diluent: None  
Diluent producer / no.: None / None  
Dilution factor: 1:0  
Incubation time / temperature: 30 min. / 26°C

### Epitope retrieval, HIER

Device: Other  
Buffer: Fuzhou Maixin DNS  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 99°C

### Visualization system

Producer: Fuzhou Maixin  
Product / no: Titan super / TT-0805  
Linker: super  
Incubation time linker: 13 min.  
Incubation time polymer: 13 min.  
Incubation temperature: 26°C

### Chromogen

Producer: Fuzhou Maixin  
Product / no: DAB / TT-0805  
Incubation time / temperature: 6 min. / 26°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.