

Recommended protocol for CK8/18

Obtained in run 69

04 Jul 2023

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: EP17/EP30
Producer: Dako
Product no. / lot no.: M3652 / 11120072
Diluent: Antibody Diluent
Dilution factor: 1:50
Incubation time / temperature: 32 min. / 37°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana Ultra CC1
Heating time at max. temp.: 64 min.
Maximum heating temp.: 96°C

Visualization system

Producer: Ventana
Product / no: UltraView Universal DAB Detection Kit / 760-500
Incubation time polymer: 8 min.
Incubation temperature: 37°C

Chromogen

Producer: Ventana
Product / no: ultraView Universal DAB Detection Kit / 760-500
Incubation time / temperature: 8 min. / 37°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.