

## Recommended protocol for CK8/18

Obtained in run 69

28 Sep 2023

### Immunostainer

Type: Dako Autostainer Link 48 +

### Primary antibody

Clone: EP17/EP30  
Producer: Dako  
Product no. / lot no.: M3652 / 11566443  
Diluent: Antibody Diluent  
Dilution factor: 1:50  
Incubation time / temperature: 20 min. / 20°C

### Visualization system

Producer: Dako  
Product / no: EnVision FLEX+ / K8002/SM802  
Linker: Linker, Rabbit  
Incubation time linker: 15 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 20°C

### Chromogen

Producer: Dako  
Product / no: DAB+ / DM827 / DM823 / K800  
Incubation time / temperature: 10 min. / 20°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.