

Recommended protocol for CD138

Obtained in run 69

11 Oct 2023

Immunostainer

Type: Thermo Autostainer 36/48/72

Primary antibody

Clone: B-A38
Producer: Immunologic
Product no. / lot no.: ILM3825-C1 / 060721-A
Diluent: Bright diluent
Diluent producer / no.: Immunologic / UD09-999
Dilution factor: 1:400
Incubation time / temperature: 60 min. / 25°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 99°C

Visualization system

Producer: Immunologic
Product / no: Brightvision / DPVB999HRP-polymer
Linker: None
Incubation time polymer: 30 min.
Incubation temperature: 25°C

Chromogen

Producer: Immunologic
Product / no: Bright Dab Solution / UBVB999HRP-BS04A
Incubation time / temperature: 8 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.