

Recommended protocol for MLH1

Obtained in run 49

03 Jan 2017

Immunostainer

Type: Dako Omnis

Primary antibody

Clone: ES05
Producer: Leica/novocastra
Product no. / lot no.: NCL-L-MLH1 / 6040968
Diluent: renoir red
Dilution factor: 1:75
Incubation time / temperature: 32 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Dako Omnis Target Retrieval Solution, High pH
Heating time at max. temp.: 40 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako Omnis
Product / no: EnVision Flex / GV800/GV823
Linker: Mouse LINKER
Incubation time linker: 10 min.
Incubation time polymer: 30 min.
Incubation temperature: 24°C

Chromogen

Producer: Dako Omnis
Product / no: DAB+ Substrate Chromogen System / GV825
Incubation time / temperature: 5 min. / 24°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.