

Recommended protocol for NKX3.1

Obtained in run 49

09 Jan 2017

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: 6E7
Producer: Origene
Product no. / lot no.: UMAB196 / TA805425
Diluent: Normal Antibody Diluent
Diluent producer / no.: Immunologic / 38243
Dilution factor: 1:1300
Incubation time / temperature: 32 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana Ultra CC2
Heating time at max. temp.: 68 min.
Maximum heating temp.: 91°C

Visualization system

Producer: Ventana
Product / no: UltraView Universal DAB Detection Kit / 760-500
Incubation time polymer: 8 min.
Amplifier: Ultraview/iView Amplification Kit
Incubation time amplifier: 16 min.
Incubation temperature: 36°C

Chromogen

Producer: Ventana
Product / no: ultraView Universal DAB Detection Kit / 760-500
Incubation time / temperature: 8 min. / 36°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.