

Recommended protocol for Bcl-6

Obtained in run 70

18 Dec 2023

Immunostainer

Type: Gene Tech GeneStainer

Primary antibody

Clone: LN22
Producer: Leica/novocastra
Product no. / lot no.: NCL-L-Bcl-6-564 / 6082532
Diluent: Ab diluent
Diluent producer / no.: Gene Tech / GT100910
Dilution factor: 1:50
Incubation time / temperature: 50 min. / 25°C

Epitope retrieval, HIER

Device: Water bath
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 95°C

Visualization system

Producer: Gene Tech
Product / no: GTVision / GK8007
Linker: None
Incubation time polymer: 20 min.
Incubation temperature: 25°C

Chromogen

Producer: Gene Tech
Product / no: DAB / GK8007
Incubation time / temperature: 5 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.