

## Recommended protocol for GATA3

Obtained in run 70

28 Dec 2023

### Immunostainer

Type: Leica BOND III

### Primary antibody

Clone: L50-823  
Producer: Cell Marque  
Product no. / lot no.: 390M-14/15/16 / 177852  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / K8006  
Dilution factor: 1:100  
Incubation time / temperature: 15 min. / 22°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Leica Bond Epitope Retrieval Solution 2  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 100°C

### Visualization system

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Incubation temperature: 22°C

### Chromogen

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time / temperature: 10 min. / 22°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.