

## Recommended protocol for CD20

Obtained in run 71

15 Apr 2024

### Immunostainer

Type: Dako Omnis

### Primary antibody

Clone: L26  
Producer: Dako/Agilent  
Product no. / lot no.: M0755 / 41597660  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / K8006  
Dilution factor: 1:200  
Incubation time / temperature: 30 min. / 36°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Epredia Dewax and HIER Buffer H  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 97°C

### Visualization system

Producer: Dako Omnis  
Product / no: EnVision Flex / GV800/GV823  
Linker: Mouse LINKER  
Incubation time linker: 10 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 36°C

### Chromogen

Producer: Dako Omnis  
Product / no: DAB+ Substrate Chromogen System / GV825  
Incubation time / temperature: 5 min. / 36°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.