

Recommended protocol for CD20

Obtained in run 71

15 Feb 2024

Immunostainer

Type: MDStainer-Vitro

Primary antibody

Clone: L26
Producer: Master Diagnostica
Product no. / lot no.: MAD-002037QD/V / 20370053DS
Diluent: None
Diluent producer / no.: None / None
Dilution factor: 1:1
Incubation time / temperature: 25 min. / 25°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Master Diagnostica TR1 high pH solution
Heating time at max. temp.: 30 min.
Maximum heating temp.: 101°C

Visualization system

Producer: Master Diagnostica
Product / no: Master Polymer Plus Detection System (peroxidase) / MA
Linker: Primary Antibodies Amplifier Master
Incubation time linker: 13 min.
Incubation time polymer: 15 min.
Amplifier: Master Polymer Plus HRP
Incubation time amplifier: NA min.
Incubation temperature: 25°C

Chromogen

Producer: Master Diagnostica
Product / no: Master Polymer Plus Detection System (peroxidase) / MA
Incubation time / temperature: 10 min. / 25°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided



Nordic Immunohistochemical Quality Control

Institute of Pathology, Aalborg University Hospital, Ladegaardsgade 3, P.O.Box 561, DK-9100 Aalborg, Denmark

information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.